# Flairesse

## Prophylaxis gel

DMG Chemisch-Pharmazeutische Fabrik GmbH  
Elbgaustraße 248 · 22547 Hamburg · Germany  
www.dmg-dental.com  
091692/#3\_2016-01

<b>DE</b>	Gebrauchsinformation	<b>SV</b>	Bruksanvisning
<b>EN</b>	Instructions for use	<b>PL</b>	Instrukcja użycia
<b>FR</b>	Mode d'emploi	<b>RU</b>	Информация по применению
<b>IT</b>	Istruzioni per l'uso		
<b>ES</b>	Instrucciones de uso		
<b>PT</b>	Instruções de utilização		
<b>NL</b>	Gebruiksaanwijzing		
<b>DA</b>	Brugsanvisning		

### Gebrauchsinformation

### Deutsch

#### Produktbeschreibung

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## English

### Product description

Prophylaxis gel for final treatment as part of professional tooth cleaning (PTC) and for integration into an individual prevention plan. The prophylaxis gel is available in three different flavors.

### Indications

- Promotes remineralization of enamel after professional tooth cleaning.
- Remineralization of initial caries.

### Contra-indications

Do not use the material

- In known allergies against any of the substances or contact dermatitis,
- Or if the patient's swallowing reflex is not guaranteed.

### Basic safety instructions

- For dental use only.
- Keep out of reach of children.
- Do not swallow!
- Avoid contact with the eyes! In the event of inadvertent contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.

### Side effects

Swallowing excessive amounts of the material may cause nausea.

### Dosage

Max. 4 ml (4 g) gel per treatment.

### Instructions for use

- ▶ The bottle has a tamper-proof seal. Make sure that the seal is intact before first use. When using the bottle for the first time, unscrew the cap, remove the tamper-proof seal by pulling the tag and discard.
- ▶ Shake well before each use!

### Recommended use

**Caution!** Risk of swallowing if saliva suction insufficient.

- ▶ Saliva must be suctioned thoroughly during treatment.

### Applying the gel with a polishing cup

**Caution!** Incorrect use may result in overheating.

- ▶ Ensure that there is sufficient material in the polishing cup used.

1. Open the bottle cap.
2. Hold the bottle vertically with the opening pointing down and express the gel into a dappen glass by gently squeezing the bottle.
3. Dry the tooth surfaces.
4. To fill, hold the idle polishing cup in the prophylaxis gel and pick up the material.

**Please note:** Any commercially available polishing cup may be used.

5. Work in the prophylaxis gel using the polishing cup.
6. Get the patient to spit out the excess prophylaxis gel.

**Please note:** For guaranteed effectiveness, inform the patient that they should not do the following for 30 minutes after using the prophylaxis gel:

- Eat
- Drink
- Rinse their mouth.

### Applying the gel with an application tray

1. Open the bottle cap.
2. Hold the bottle vertically with the opening pointing down and express the gel into the application tray by gently squeezing the bottle.
3. Dry the tooth surfaces.
4. Apply the application tray to the dental arch.
5. Leave the application tray on the dental arch for 1 minute.
6. Remove the application tray at the end of the response time.
7. Get the patient to spit out the excess prophylaxis gel.

**Please note:** For guaranteed effectiveness, inform the patient that they should not do the following for 30 minutes after using the prophylaxis gel:

- Eat
- Drink
- Rinse their mouth.

### Ingredients

Water, o-phosphoric acid (< 3 %), flavors and scents, additives, Sodium fluoride (2.72 % corresponds to 1.23 % fluoride ions/12.300 ppm).

Free from aspartame, gluten, saccharine.

Contains Xylitol.

### Storage and shelf life

- Store at room temperature (20 to 25 °C/68 to 77 °F)!
- Do not store in the refrigerator!
- Protect against exposure to direct sunlight!
- Do not use after the expiration date!

### Available product sizes

Packs contain a 480 ml bottle and are available in the following flavors:

Mint	REF 220411
------	------------

Melon	REF 220412
Strawberry	REF 220413

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