# Flairesse

## Prophylaxis varnish

DMG Chemisch-Pharmazeutische Fabrik GmbH  
Elbgaustraße 248 · 22547 Hamburg · Germany  
www.dmg-dental.com  
091690/#4\_2016-01

<b>DE</b>	Gebrauchsinformation	<b>SV</b>	Bruksanvisning
<b>EN</b>	Instructions for use	<b>PL</b>	Instrukcja użycia
<b>FR</b>	Mode d'emploi	<b>RU</b>	Информация по применению
<b>IT</b>	Istruzioni per l'uso		
<b>ES</b>	Instrucciones de uso		
<b>PT</b>	Instruções de utilização		
<b>NL</b>	Gebruiksaanwijzing		
<b>DA</b>	Brugsanvisning		

### Gebrauchsinformation

### Deutsch

#### Produktbeschreibung

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## English

### Product description

Moisture and saliva tolerant varnish for treating hypersensitive teeth. Applied to the enamel and dentine as part of prophylactic treatment. Flairese prophylaxis varnish comes in a Unit Dose and is available in two different flavors.

### Indications

- Treatment of hypersensitive teeth.
- Sealing of dentin tubules after cavity preparations.
- Sealing of hypersensitive areas at the neck of the tooth and exposed root dentine.
- Protective varnish for preventing caries.
- Remineralisation of initial caries.

### Contra-indications

Do not use the material

- In cases of allergies to any of the ingredients or in cases of contact allergies,
- If the patient's control over their swallowing reflex is not guaranteed,
- If the patient's gums are ulcerated or
- If the patient is suffering from stomatitis.

### Basic safety instructions

- For dental use only.
- Keep out of reach of children.
- Do not swallow.
- Avoid contact with the eyes! In the event of inadvertent contact with the eyes, immediately rinse the eyes thoroughly with plenty of water and consult a physician if necessary.

### Side effects

- In rare cases, use of the product may lead to breathing difficulties in patients suffering from asthma.
- In rare cases, extensive application may lead to oedematous swelling of the gums.
- Swallowing excessive amounts of the material may cause nausea.

### Interactions

- Other prescription-only products containing fluoride should not be used within 24 hours of using the product.
- Patients using regular, systemic fluoridation should stop using these products for 2 or 3 days following use of this product.

### Advice on application

- ▶ The Unit Dose is fitted with a tamper-proof seal. Make sure that the tamper-proof seal is intact before first use.

### Recommended use

**Please note:** Each Unit Dose holds enough material for one prophylaxis treatment. Each Unit Dose may only be used for one individual patient.

1. Remove calculus and plaque.
2. Remove saliva and fluid from the area to be treated.
3. Peel off the tamper-proof seal from the Unit Dose using the tag and discard.
4. Remove the brush and use it to carefully mix the varnish.
5. Bend the brush as required for treatment.
6. Apply a thin, even coating of the varnish to the areas to be treated.
7. Allow to work in for 4 hours.

**Please note:** In order to guarantee the response time of 4 hours, inform the patient that they may not do any of the following for this period:

- Eat any solid food
- Drink any alcoholic drinks
- Use any alcohol-based mouthwash
- Brush their teeth
- Use dental floss.

### Ingredients

Natural resin, ethanol, flavors and scents, additives, sodium fluoride (5 %, corresponds to 2.26 % fluoride ions/22.600 ppm).

Free from aspartame, gluten, saccharine.

Contains Xylitol.

Contains rosin.

### Storage and shelf life

- Store at room temperature (20 to 25 °C/68 to 77 °F).
- Do not store in the refrigerator.
- Protect against exposure to direct sunlight.
- Do not use after the expiration date.

### Available product sizes

#### Unit Dose

One pack contains 35 individual portions @ 0.40 ml with application brush. Available in the following flavors:

Mint	REF 220404
Melon	REF 220405

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