# PermaCem

## Inhalt | Contents

Gebrauchsinformation – DEUTSCH	4
Instructions for use – ENGLISH	11
Mode d'emploi – FRANÇAIS	18
Istruzioni per l'uso – ITALIANO	26
Instrucciones de uso – ESPAÑOL	34
Instruções de utilização – PORTUGUÊS	41
Gebruiksaanwijzing – NEDERLANDS	48
Brugsanvisning – DANSK	55
Bruksanvisning – SVENSKA	62
Instrukcja użycia – POLSKI	69
Информация по применению – РУССКИЙ	77

## Produktbeschreibung

---

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## Product Description

---

PermaCem is a universal, adhesive, self-curing compomer dental cement for cementing indirect restorations in combination with an enamel-dentine bonding agent such as LuxaBond (total-etch system) or Contax + Activator (self-etch system).

## Indications

---

Permanent, adhesive cementation of crowns, bridges, inlays, onlays and veneers composed of

- Metal,
- Ceramic,
- Composite.

## Contra-indications

---

- The use of PermaCem with dentine single-bottle bonding agents is contra-indicated.
- Do not use PermaCem in cases of known allergies to any of the substances contained

or, as can occur in rare cases, in the event of contact allergy.

- The use of PermaCem with oxide ceramics is contra-indicated if no parallel-wall preparation is possible.

## Basic Safety Instructions

---

- ▶ For dental use only!
- ▶ Keep out of reach of children!
- ▶ Avoid contact with the skin! In the event of inadvertent skin contact, wash the affected area thoroughly with soap and water.
- ▶ Avoid contact with eyes! In the event of inadvertent contact with eyes, immediately rinse the eyes thoroughly with plenty of water and consult a physician if necessary.
- ▶ Cover dentine close to the pulp with pulp protection. If applicable, apply pulp protection before creating an impression.
- ▶ In the event of allergic reactions, remove the material immediately and consult a physician, if necessary.

## Side effects

---

There have been no systemic side effects to date. Isolated cases of hypersensitivity to components of the material cannot be excluded.

## Interactions with other materials

---

Do not use PermaCem together with materials containing eugenol. DMG recommends eugenol-free temporary cements (e.g. TempoCemNE).

## Application systems

---

- ▶ PermaCem Automix: see “Using the Automix cartridge” on page 14.
- ▶ PermaCem Smartmix: see “Using the Smartmix syringe” on page 15.

## Timing

---

Working time (including mixing time)*	≈ 2 min
Intraoral curing	5 min

\*The time stated applies at a room temperature of 23 °C. Higher temperatures may shorten these times, while lower temperatures may increase them..

## Instructions for use

---

Cementation before combined work such as telescope work: Using a suitable separating agent, ensure that PermaCem cannot come between primary and secondary parts.

## Recommended Use

---

1. Prepare the clean and dry restoration as follows:
  - ▶ Etchable silicate ceramic: Etch the restoration with 5% hydrofluoric acid in accordance with the instructions from the manufacturer and rinse thoroughly with water spray.


Subsequently dry with oil- and water-free air, and pre-treat the surface using a suitable silanizing agent (e.g. DMG Silane) in accordance with the manufacturer's instructions.

- ▶ Metals and metal alloys: Pretreatment in accordance with the manufacturer's instructions. Unless otherwise stated, sandblast the inside of the restoration with aluminum oxide, clean with alcohol and dry with oil- and water-free air.
  - ▶ Oxide ceramics: Pretreatment in accordance with the manufacturer's instructions. Ensure that there is sufficient parallel-wall retention.
2. Remove any residue of the temporary cement from the prepared tooth, clean thoroughly with water spray and dry with oil- and water-free air. When doing so, ensure that the dentin retains a little residual moisture and does not become overly dry.

**Note:** The use of rubber dam is recommended.

3. Apply a suitable bonding agent to the prepared tooth according to manufacturer's instructions. DMG recommends LuxaBond (total etch system) or Contax + Activator (self-etch system).
4. Apply PermaCem in the pre-treated restoration or on the prepared tooth .
5. Insert the restoration within 2:00 min of the start of mixing and attach it to the prepared tooth using slight pressure.
6. Instruct the patient to adopt habitual occlusion.
7. Remove excess cement within 1:30 to 2:00 min of inserting the restoration with the aid of a microbrush, brush, foam pellet or scaler. Remove interdental excess by flossing carefully.
8. Leave material to cure. After 5:00 minutes, the restoration has fully regained its capacity to withstand stress.

## Using the Automix cartridge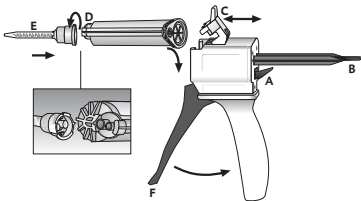

### Insert cartridge

1. Push up the lever [A] at the back of the Automix-Dispenser and pull back the slide [B] completely.
2. Lift up the plastic lever [C], insert the cartridge and lock it using the plastic lever [C].

**Note:** Make sure that the notches on the cartridge line up with those on the Automix-Dispenser.

### Attach mixing tip

1. Remove the cartridge cap [D] or used mixing tip by turning it 90° counterclockwise, and discard it.
2. Attach a new mixing tip [E].

**Note:** In order to attain an optimum mixing result, DMG recommends the use of the mixing tips available from DMG. All mixing tips are also available as refill packs.

**Note:** Make sure that the notches on the mixing tip line up with those on the cartridge.

3. Turn the mixing tip clockwise 90° to lock it.

### Applying the material

- ▶ When using a cartridge for the first time, squeeze out a pea-sized amount of material and discard it.
- ▶ The material is mixed in the mixing tip by pulling the trigger [F] of the Automix-Dispenser and it is then applied directly.


**Note:** After the application, leave the used mixing tip on the cartridge to cap unused material!

### Remove cartridge


1. Push up the lever [A] at the back of the Automix-Dispenser and pull back the slide [B] completely.
2. Lift up the plastic lever [C] and remove the cartridge.

### Using the Smartmix syringe

---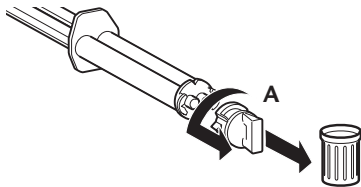

1. Before attaching the mixing tip, remove the cap [A] or used mixing tip after turning it 90° counterclockwise, and discard it.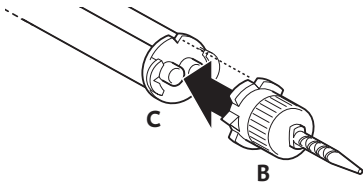


2. Attach a new mixing tip [B].

**Note:** In order to attain an optimum mixing result, DMG recommends the use of the mixing tips available from DMG. All mixing tips are also available as refill packs.

**Note:** Make sure that the notches on the mixing tip line up with those on the Smartmix injector [C].

3. Turn the mixing tip clockwise 90° to lock it.

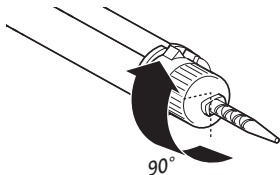4. The material is mixed in the cannula when it is expressed and it can be applied directly.

**Note:** When using an injector for the first time, squeeze out a pea-sized amount of material and discard it!

**Note:** Leave the used mixing cannula on the Smartmix syringe after use. It serves as a cap.

## Disinfection

---

- The Automix-Dispenser is reusable after disinfection, although it should be replaced if visible signs of wear appear. Disinfection can be carried out using a commercially available dipping disinfectant. The Automix-Dispenser is also autoclavable.
- Dirty or contaminated mixing cannulas used as a cap can be cleaned with disinfectants normally used in practice.
- DMG recommends using only disinfecting solutions listed by the RKI (Robert Koch Institute).

## Storage and shelf life

---

- ▶ Store at 2 – 25 °C/36 – 77 °F in a dry place.
- ▶ Do not use after the expiration date.

## Composition

---

Ionomer glass in a Bis-GMA-based matrix consisting of dental resins, activators and additives.

## Packaging

PermaCem-Smartmix	
2 syringes containing 10 g paste 20 Smartmix-Tips	REF 212034*

PermaCem-Automix	
1 cartridge containing 52 g paste 35 Automix-Tips	REF 110293*

\* Not available in the USA and Canada

Accessories	
1 Automix-Dispenser type 25 1:1	REF 110253
50 Automix-Tips Blue 1:1	REF 110266
100 Automix-Tips Blue 1:1	REF 110267
50 Smartmix-Tips Short	REF 212040

## Only available in the United States and Canada

Starter pack	
1 cartridge containing 52 g PermaCem-Automix, 1 Automix-Dispenser, 50 Automix-Tips	REF 110292

Refill pack	
1 cartridge containing 52 g PermaCem-Automix, 40 Automix-Tips	REF 110472

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**DMG** Chemisch-Pharmazeutische Fabrik GmbH