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4 g

11035

DIRECTIONS FOR USE

GB

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Calcium hydroxide, light-curing, radiopaque paste for cavity lining

Composition

Urethane dimethacrylate, Calcium Hydroxide, Barium Sulfate, Silicates, excipient

Properties

Radiopaque, one-component paste based on Calcium Hydroxide and Dimethacrylates, light-cured through curing light.

The paste provides:

- ▶ excellent stability
- ▶ high resistance to resorption and dissolution
- ▶ excellent adhesion to various restoration materials (see below «recommendations»)

Indications

- ▶ Cavity lining under composites and amalgams
- ▶ Indirect pulp capping

Directions for use

- ▶ Prepare the cavity in the usual way and ensure that the dental surfaces are completely dried
- ▶ Place the required quantity of paste onto a mixing pad and avoid exposing to direct light (sun, lamps, etc.)
- ▶ Apply the mix with a suitable instrument, in successive layers in case of deep cavity
- ▶ Cure under curing light, following the manufacturer's instructions for use
 - In cases where the lining needs to be more than 1 mm thick, polymerize layer by layer
 - In order to ensure ideal polymerization, place the source of light as close to the surface as possible (1 to 2 mm) and polymerize for 40 to 60 seconds
 - If the distance is greater, increase the polymerization time accordingly
- ▶ Remove any excess material with a rotating instrument (hand instruments could cause the removal of the whole lining) and seal

Recommendations

- ▶ To prevent leakage, apply permanent lining to the cervical and approximal margins
- ▶ Directly next to the pulp, apply an aqueous solution based on Calcium Hydroxide
- ▶ Light-curing paste adheres to various materials (particularly those based on methacrylates or polyether). In order to protect the lining, an isolating agent, like vaseline, should be applied and removed prior to permanent obturation
- ▶ Avoid all contact and/or cross-contamination with materials containing eugenol or thymol which may inhibit polymerization
- ▶ This product contains methacrylates, BHT and amine. Do not apply the material to patients who have known allergies to these ingredients

Warning

This product is made of calcium hydroxide, a skin irritant which can cause serious damage to the eye. When using this product, gloves and protective glasses should be worn.

In case of contact with the skin wash with plenty of soap and water. If skin irritation occurs, get medical advice/attention. In case of contact with the eye rinse cautiously with soft, clean water for 15 minutes, holding the eyelids open, and call a doctor immediately. In case of swallowing rinse the mouth with water and consult a doctor. Keep the person exposed at rest, do not force vomiting.

Storage

Store at room temperature between 4°C and 23°C, protected from direct light and humidity.

Packaging

Syringe of 4g + 5 nozzles

For dental professional use. Made in Switzerland.

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