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DIRECTIONS FOR USE

Copal Varnish

Protective dental varnish for the insulation of the exposed dentine

Composition

Damar Gum 2.4%, Rosin 2.9%,
Linseed Oil Resins 11.75%,
Chlorobutanol 0.1%,
not chlorinated Solvent 100%

Properties and Indications

Copal Varnish is the ideal cavity liner and dental tubuli sealer. It ensures perfect protection to exposed dental surfaces due to be sealed with either permanent fillings or restorations.

It is also useful as an intermediary varnish prior to the insertion of amalgams, Silicate or Oxyphosphate cements as it offers a safeguard against irritant acids. Copal Varnish hardens instantly forming a thin coating. The use of successive thin layers is more effective than using just one layer of thicker varnish.

Directions for use

- > Isolate the tooth with a rubber dam or cotton rolls
- > Dry the cavity thoroughly with alcohol and an air-dryer
- > Use a cotton pellet dipped in the varnish to rapidly coat the exposed dentine
- > Dry the coat of varnish with an air-dryer
- > Repeat the operation until the surface shows a slight glaze after evaporation of the solvent.
To prevent contamination of the bottle, use a new cotton pellet for each application.
The number of coats of varnish is determined by the porosity of the dentine and the slight glaze indicates suitable saturation
- > After the last coat of varnish, dry thoroughly
- > Proceed with the filling or the restoration

Precaution

The rates of evaporation of the different elements which compose the solvent are not identical.

In order to avoid any undue alteration of Copal Varnish, leftover solvent should not be used to top up the bottle.

The integrity of the film is destroyed in contact of composite restorative materials.

Storage

Store at room temperature between 15°C and 25°C, protected from direct light and, above all, from heat or flame.

Flammable liquids (varnish and solvent).

Packaging

Portion of 15 ml varnish and 15 ml solvent
Bottle of 15 ml of varnish only
Bottle of 15 ml of solvent only

REF

11040

11041

11042

- > FOR DENTAL PROFESSIONAL USE
- > MADE IN SWITZERLAND

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