Mummifying Paste

Antiseptic, non resorbable and radiopaque root canal paste

Composition

Polyoxymethylene 20%, Cresol 15%, radiopaque excipient ad 100%

Properties

Mummifying Paste is a non resorbable and radiopaque preparation for the temporary or the permanent obturation of the root canal, to be used when a powerful, efficient antiseptic treatment is required against aerobic and/or anaerobic bacteria. This preparation does not contain any anti-inflammatory component which, in some cases, could be contra-indicated. Consequently, there may be slight risk of arthritis if the paste were to protrude the apex. Where an anti-inflammatory agent is required, Endofill with Dexamethasone is recommended.

Indications

Temporary or permanent treatment and obturation of the root canal to be used more specifically in cases where it is not possible to totally remove the pulp.

Directions for use

- ▶ Devitalize and remove the radicular pulp as far as possible, rinse and dry the canal
- ▶ Use a probe or a rotary paste filler to insert the paste into the root canal
- ▶ Avoid air pockets or piercing the apex

Temporary treatment

Cover with a temporary cement and re-open after 5 – 7 days. Remove the paste mechanically (if necessary use Eugenate Desobturator to soften the paste). Clean the root canal thoroughly and complete the obturation or the restoration as desired.

Permanent obturation

Seal the canal with Zinc Phosphate Cement and complete the restoration as desired

Warning

Contains Polyoxymethylene (Paraformaldehyde). Toxic by inhalation and ingestion. When using this product, gloves and protective glasses should be worn. In case of contact with the skin wash with plenty of soap and water. If skin irritation occurs, get medical advice/attention. In case of contact with the eye rinse cautiously with soft, clean water for 15 minutes, holding the eyelids open, and call a doctor immediately. In case of swallowing rinse the mouth with water, drink a lot of water and consult a doctor. Keep the person exposed at rest, do not force vomiting. Do not use in cases of known hypersensitivities to any of these ingredients.

Storage

Store at room temperature between 5°C and 30°C, protected from direct light and humidity. Close the jar carefully after use.

Shelf life 3 years.

Packaging

Jar of 12g

 11560

For dental professional use. Made in Switzerland.

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©PD 140210