# Silagum-Comfort Primer & Varnish

DMG Chemisch-Pharmazeutische Fabrik GmbH
Elbgaustraße 248 · 22547 Hamburg · www.dmg-dental.com
091753/#1\_2012-03

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**Produktbeschreibung**

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<b>Instructions for Use</b>	<b>English</b>
<b>Product description</b>	

Silagum-Comfort Primer is a bonding agent for Silagum-Comfort. Silagum-Comfort Varnish is used for post-processing of a prosthesis relined with Silagum-Comfort to create an even surface.

**Indications**

- Silagum-Comfort Primer: Bonding between standard PMMA-based prosthesis materials and Silagum-Comfort
- Silagum-Comfort Varnish: Post-processing of a prosthesis relined with Silagum-Comfort

**Basic safety instructions**

- For dental use only!
- Keep out of reach of children!

**Recommended use**

**Applying Silagum-Comfort Primer**

- Apply Silagum-Comfort Primer to the surface of the prepared prosthesis and allow the solvent to completely evaporate (approx. 1 min). Ensure that the primer is applied thoroughly to the areas at the edges. Apply at least two layers.
- Close the bottle tightly to prevent the solvent from evaporating.**Applying Silagum-Comfort Varnish**

- Mix Silagum-Comfort Varnish evenly at a ratio of 1:1.
- Using a brush, apply Silagum-Comfort Varnish to the prepared areas of the prosthesis within the working time, avoiding air bubbles as much as possible.
- Allow Silagum-Comfort Varnish to dry for 15 min.

	<b>Silagum-Comfort Primer</b>	<b>Silagum-Comfort Varnish</b>
Working time*	-	≤ 2 min
Hardening time*	-	≥ 15 min
Airing time	≥ 1 min	-

\* The times apply at a room temperature of 23 °C and a normal relative air humidity of 50%. Higher temperatures shorten, lower temperatures increase these times.

**Disinfection**

The brush holder is reusable after disinfection, although it should be replaced if visible signs of wear, e.g. scratches, appear. Disinfection can be carried out using a commercially available wiping or dipping disinfectant. DMG recommends using only disinfecting solutions listed by the RKI (Robert Koch Institute).

**Storage and shelf life**

- Store at room temperature (15 – 25 °C/59 – 77 °F) in a dry place.
- Do not store in the fridge or freezer.
- Protect against exposure to direct sunlight.
- Do not use after the expiration date.

**Composition**

Silagum-Comfort Primer: Ethyl acetate, modified polyacrylate, additive
Silagum-Comfort Varnish: Vinyl silicone, hydrogen silicone, additive, platinum cata-lyst

**Available product sizes**

Silagum-Comfort Primer [1 bottle @ 5 ml]	REF 909090
Silagum-Comfort Varnish [2 bottles @ 10 ml]	REF 909089

Federal law (USA) restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

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